

lion alcolmeter® SD-400



...a breath alcohol instrument for field use - anywhere!
...fast ...simple ...accurate!

Instrument Features

- **SIMPLE TO USE:** the lion alcolmeter® SD-400 is fully automatic. This means it's very simple to use, so training requirements are minimal.
- **FAST, ACCURATE ANALYSIS:** using the Lion *fuel cell* sensor. This ensures high specificity to alcohol, and a long stable working life.
- **EASY TO READ DISPLAY:** alcohol readings are shown on a large illuminated display. Four indicator lights inform and instruct the operator.
- **LAST TEST RECALL:** the result of the last breath test can be recalled, even if the instrument has since been switched off.
- **AUTOMATIC DATA COLLECTION:** all alcohol readings are stored in memory, with date and time, and can be downloaded to a printer or PC. An optional data-logging system [eg operator ID, subject sex, test location] is also available.
- **TRUE EVIDENTIAL CAPABILITY:** the lion alcolmeter® SD-400 may be used for fast subject screening, but is sufficiently accurate for *evidential* testing. Attach the battery-operated printer [SD-400P] or use a PC with OnLine-400™ for data input, automated control and comprehensive documentation.
- **CALIBRATION SECURITY:** by means of the 'dongle' electronic key.
- **APPROVALS:** the instrument is Approved by law enforcement authorities worldwide, for both screening *and* evidential breath testing applications.




Simple Operating Procedure!



1
SWITCH ON:
AUTOMATIC SELF CHECK



2
ATTACH MOUTHPIECE:
INSTRUCT THE SUBJECT

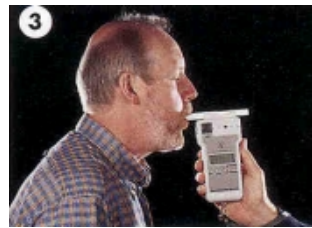

3
THE SUBJECT BLOWS



4
NOTE ALCOHOL READING

Advanced Specifications and Features

Fuel Cell Sensor: high specificity to alcohol, unaffected by other possible breath contaminants.

Warm-up Time: the instrument is ready for use within 15 seconds of switch on, at any, temperature.

Measuring Range: normally 0.02 to 2.00mg/iitre BRAC. An overrange indication is given at higher levels.

Analytical Response Time: normally within 30 seconds; but within 5 seconds at zero alcohol levels.

Temperature Range: from -5°C to +40°C, for optimum operation.

Recalibration: every six months, using either a wet-bath simulator or Alcolac[®] gas standard.

Recovery Time: typically 10 seconds at zero alcohol; 60 seconds at 0.35mg/iitre BRAC; 4 minutes at 2.00mg/iitre.

Memory Capacity: 500 tests as standard, expendable up to 5,000 on demand.

Power Supply: 5 new AA cells give at least 3,000 breath tests. Two stage low power warning system.

Radio Frequency Immunity: exceeds requirement of EN 50082-1.

Shock/Vibration: impact resistant ABS: meets IEC 68-2-27 Part 2.1 Test EA and IEC 68-2-6 Part 2.1 Test FC.

Dimensions: 169 x 80 x 39mm [instrument only]; 190 x 90 x 65mm [in leather pouch].

Weight: approximately 525g [including batteries and leather pouch].

Date and Time: each test result is stored with date and time [internal battery backed].

Accessory Items

- **Disposable Mouthpieces:** low resistance to breath flow; individually and hygienically wrapped.
- **Reusable Sampling Cup:** an alternative to the mouthpiece, for rapid subject scimpling and turnover, such as at a Police roadblock or factory gate.
- **Protective Pouch:** in durable grey or reflective yellow leather.
- **Rechargeable Power Unit:** with 220 volts AC/12 volts DC intelligent charger.
- **Vehicle Power Unit:** powers the 400 from a vehicle's 12 volt DC supply [via the cigarette lighter socket].
- **Data Printer:** prints stored data from memory.
- **Printer Upgrade Kit:** convert to the SD-400P, with automatic on-line printing for every breath test.
- **Handbooks:** covering all aspects of use of the instrument and accessories.

Accessory PC Software

- **Data-400™ :** retrieve data from memory and store on your PC's hard disk; then export to any standard database or spreadsheet program. Operates under MS Windows.
- **DataMac-400™ :** the same application as Data-400", but for use in Apple Macintosh PC systems.
- **Online-400™ :** use the 400 with a PC for fully evidential documentation and printout
- **OnlineMac-400™ :** the same evidential application as Online-400'-, but for use in Apple Macintosh PC's.
- All 400 software is supplied with PC connector cable and full instructions.



INVESTOR IN PEOPLE



01151
011551



Pacific Data Systems Pty Ltd

Queensland Office

PO Box 324

Salisbury QLD 4107

TELEPHONE: (+617) 3275 2999 FAX: (+617) 3275 2244

New South Wales Office

PO Box 65

Woodford NSW 2778

TELEPHONE: (+612) 4753 1046 FAX: (+612) 4753 1213

WEB: www.pacdatasys.com.au E-MAIL: sales@pacdatasys.com.au

CE 0086
Complies with directive
93/42/EEC



FM 25502
CERTIFIED TO
INTERNATIONAL QUALITY
STANDARD ISO 9000:
EN 29000 : BS 9700